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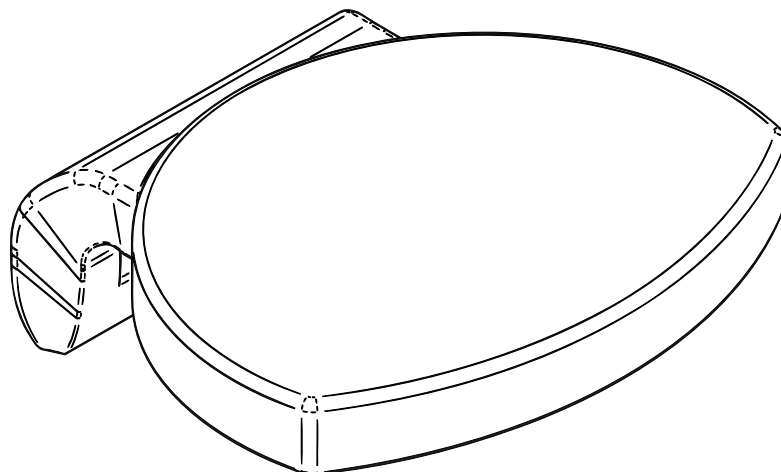
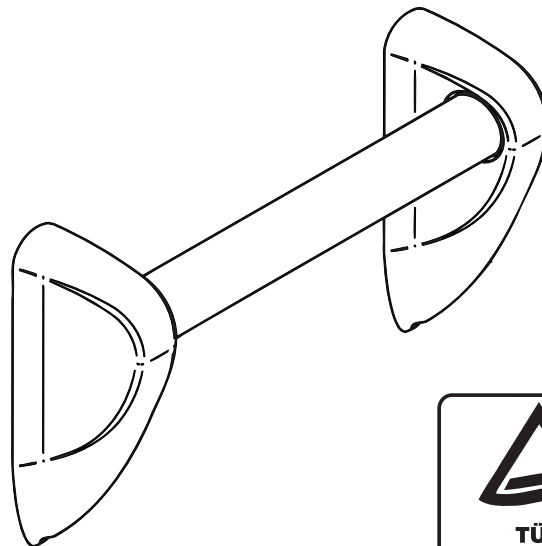
367-3 **Please pay attention to the following instructions in order to ensure an optimal functionality and long durability of your Grab Handle/Shower Seat.**

1. General

Check your shower seat for transport or other damage prior to installation. No warranty claim for damage will be accepted after installation.

2. Installation

In order to guarantee trouble free functionality of your product it is imperative to carry out the installation step by step according to the instructions. If there is any reason for complaint or if you have technical query, please do not hesitate to contact our service hotline for help.

The screws & rawlplugs provided are appropriate only for installation on to concrete walls. Please check the construction of your wall and use screws & rawlplugs which are suitable to the construction of your wall. If you are uncertain, please contact a professional plumber or builder.

3. Maintenance and care

This grab handle/shower seat consists of high quality materials. Please rinse your seat after every shower and clean it with a damp cloth. Do not use any abrasive cleaners.

anchor ground/wall work	fix up materials	comment
concrete full bricks natural stone lime sand stone full build materials	6x six edge wooden screw DIN 571-A2-8x90 6x full purpose plugs TRIKA 10/62	included in delivery of the showerseat
light build walls with back ventilation of layer wood plates or hard wood (necessary at least 20mm thickness) for example wall supports from the front wall manufacturers	6x six edge wooden screw DIN 571-A2-8x90 direct fastening without plugs	included in delivery of the showerseat
light build walls with integrated front wall installation and integrated iron plates for example wall supports from the front wall manufacturers	6x six edge wooden screw DIN 933-A2-M8x60	to set on building site !
high hole bricks hollow block stone lime sand stone other hole stones	6x six edge wooden screw DIN 571-A2-8x90 6x full purpose plugs TRIKA 10/62	included in delivery of the showerseat
pore concrete P2/P4 light concrete fullstone of light concrete V2/V4 gips wall building plates pore concrete (one corn concrete)	4x six edge screw DIN 933 - M8x80 4x anchor shell FIS H 16x85K 4x Internal screw thread anchor FIS 11 x 85 1x injection mortar FIS VS 150 C od. FIS 300 T 1x central spout PBZ 1x Konus drill PBB	to set on building site ! special retailer

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